



# HOW TO CONNECT WDL TO YOUR TRIOS SCANNER/3SHAPE COMMUNITCATE

## Woodbury Dental Lab

651-348-7848

1103 Weir Drive Suite 100

Woodbury, MN 55125

### OPTION 1:

- NAVIGATE TO THE "LAB CONNECTION PAGE" & CLICK THE CONFIGURE COMMUNICATE BUTTON.
- IN CONFIGURATION WINDOW, CLICK ADD LAB BUTTON TO LAUNCH 3SHAPE COMMUNICATE WEB PAGE.
- CLICK ON CONNECTIONS MENU: WILL OPEN CONNECTED SITE PAGE.
- CLICK ADD CONNECTION: GO TO CONNECT TO OTHER LABS
- ENTER IN LAB NAME: WOODBURY DENTAL LAB
- ENTER IN EMAIL: RAYC@WDL2.COM
- ONCE SUBMITTED WE WILL RECEIVE NOTIFICATION OF NEW REQUEST
- PLEASE CALL TO CONFIRM YOUR REQUEST: WE WILL APPROVE CONNECTION
- AFTER APPROVAL, REFRESH YOUR LAB LIST ON YOUR TRIOS SCANNER, WDL SHOULD NOW BE ON YOUR LAB CONNECTION LIST

### OPTION 2:

- SETUP NEW LAB THROUGH [3shape communicate account](#)
- SEE ATTACHED VIDEO INSTRUCTIONS :

**Trios**-Video> How to add lab connections in Trios or 3shape communicate

[https://youtu.be/NLIZrN0ygfU?si=KNsEsxHgB5OsNp\\_i](https://youtu.be/NLIZrN0ygfU?si=KNsEsxHgB5OsNp_i)

3shape 